

## TroubleShooting

In addition to the issues listed in the User's Guide, please see below for additional information.

Issue	What To Do
Settings may not save if the meter is disconnected while saving.	It is important that the meter is not disconnected during the saving operation. Some settings such as notes (custom and default) may need to be reentered and saved.
During initial set-up, Glucose Targets set on the Report Preferences menu may not be saved.	Reenter setting of the Glucose Targets in the Report Preferences menu.

## Resolución de problemas

Además de los problemas mencionados en la Guía del usuario, vea la información adicional a continuación.

Problema	Qué hacer
Puede ocurrir que no se guarde la configuración si el medidor es desconectado cuando se está guardando la misma.	Es importante que no desconecte el medidor cuando se está guardando la información. Es posible que algunas configuraciones tales como las notas (personalizadas y predeterminadas) tengan que ser reingresadas y guardadas.
Es posible que durante la configuración inicial no se guarden los Valores deseados de glucosa establecidos en el menú Preferencias de informe.	Repita la configuración de los Valores deseados de glucosa en el menú Preferencias de informe.